



Quick Start Guide



PICTURES SHOWN ARE FOR ILLUSTRATION PURPOSE ONLY

1

Download FOBO Wheely App onto your smartphone from Google Play Store (For Android) or App Store (For iOS)



Download on the
App Store




GET IT ON
Google Play

2

Ensure the Bluetooth, Location service and Internet connection on your smartphone is turned on.

Open FOBO Wheely App and sign in using your personal email address or sign in using your Facebook, Google or Apple account.

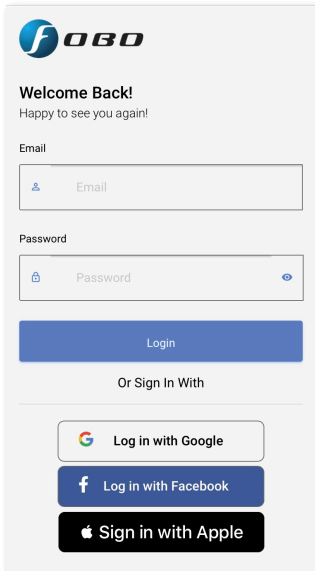


The screenshot shows the login interface of the FOBO app. At the top left is the FOBO logo, consisting of a blue circle with a white 'f' and the letters 'OBO' in a bold, black, sans-serif font. Below the logo, the text 'Welcome Back!' is displayed in a bold, black font, followed by the subtitle 'Happy to see you again!' in a smaller, regular black font. The form contains two input fields: 'Email' with a person icon on the left and 'Password' with a lock icon on the left and an eye icon on the right. A large blue button labeled 'Login' is positioned below the password field. Underneath the button is the text 'Or Sign In With'. A horizontal line separates this section from three social login options: 'Log in with Google' (with the Google logo), 'Log in with Facebook' (with the Facebook logo), and 'Sign in with Apple' (with the Apple logo).



f **OBO**

Welcome Back!
Happy to see you again!

Email


 Email


Password


 Password 

Login

Or Sign In With

 Log in with Google

 Log in with Facebook


 Sign in with Apple

4


Allow all the permissions when prompted by the App.


5


Press the plus “+” button on the FOBO Wheely App home page to start installation for a new Wheelchair.

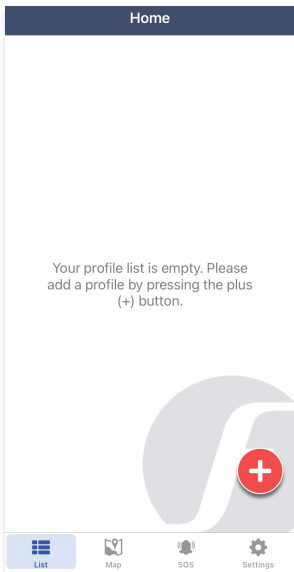


6

Select Wheelchair profile, key in the other details, press save icon  to save the profile settings.

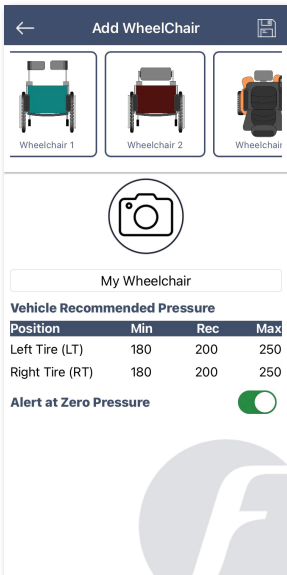

← Add WheelChair 

Wheelchair 1 Wheelchair 2 Wheelchair




My Wheelchair

Vehicle Recommended Pressure

Position	Min	Rec	Max
Left Tire (LT)	180	200	250
Right Tire (RT)	180	200	250

Alert at Zero Pressure








7

Unscrew sensor cover to remove battery tab. Then, screw back the sensor cover and ensure it is tightened.




Unscrew sensor cover

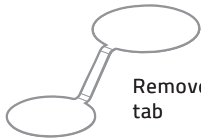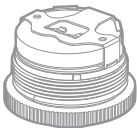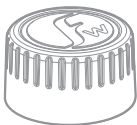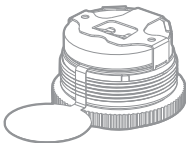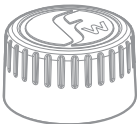

Remove battery tab

8

Follow the on-screen instructions and install the sensors to the tire valves accordingly.

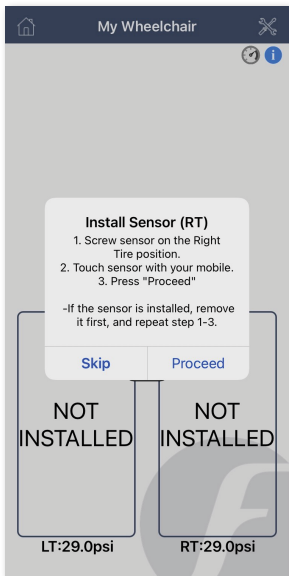

To install FOB0 Lock-Nut:

1. Screw on the FOB0 Lock-Nut to the tire valve with 'bump' facing inward until the end of the valve thread.
2. Screw on the FOB0 Wheely sensor to the tire valve securely.
3. Turn the FOB0 Lock-nut counter-clock wise as though you are removing it until it meets the FOB0 Wheely sensor.
4. Gentle hold the FOB0 Wheely sensor and use the provided wrench to further tighten the FOB0 Lock-Nut against the FOB0 sensor. This is to avoid the twisting of the rubber valve.

Note: Gently tighten the Lock-Nut against the sensor; no excessive force is required.

To Release FOB0 Wheely sensor from your smartphone:

1. Open the intended profile and long press on the tire position which you would like to release.
2. A dialog box will appear, press on Release sensor option.
3. Perform the same steps (1-2) to release the other sensors from your account.

Note: Please ensure that you have good internet connection while performing the release.

Scan here to check what's in the box.





Please download the complete user manual from
www.my-fobo.com